




**AEGIS LIFESCIENCES**

PROTECTION THROUGH PERFECTION

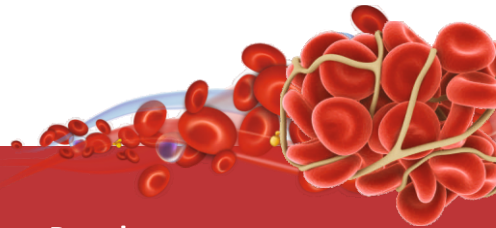

**SURGISPON<sup>®</sup>**

**Absorbable Haemostatic Gelatin Sponge Powder  
PS (Prefilled Syringe) Kit**



# SURGISPON®

## Absorbable Haemostatic Gelatin Sponge Powder PS (Prefilled Syringe) Kit



SURGISPON® Haemostatic Powder prefilled syringe kit is a sterile, absorbable Gelatin sponge powder provided in a prefilled syringe kit, intended for haemostatic use by applying to a bleeding surface especially at a difficult to reach site.

### Key Features

- Maintains uniform viscosity from beginning to end of the application after mixing with saline/thrombin
- Ready to use in 30 seconds once the sterile saline/thrombin is uniformly mixed
- Product after constitution (as directed below) envelops bleeding site to stop bleeding faster. It has a proven efficacy, safety, and efficiency across blood transfusions, major and minor complications and surgical operating time
- Stays in place during active bleeding even after mixing with saline/thrombin, and it delivers faster haemostasis where it is intended
- It is easy to use and can be applied locally at the target site of bleeding.


An empty syringe for mixing sterile solution with powder

SURGISPON® Absorbable Haemostatic Gelatin Sponge Powder

A connector luer

A liquid transfer cup

An applicator tip

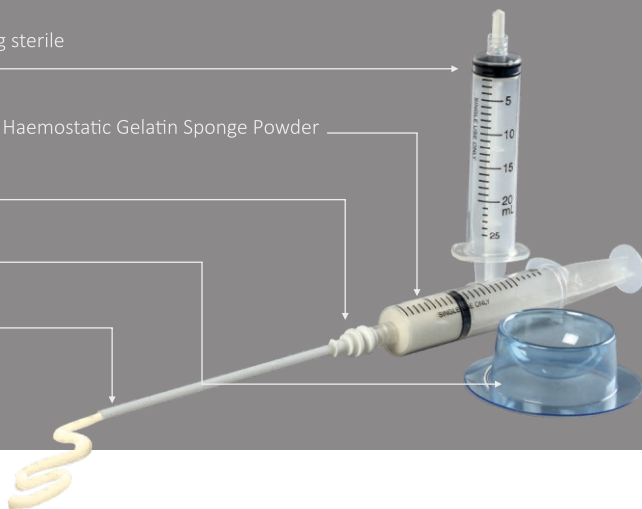

## HAEMOSTATIC MATRIX PREPARATION



**Step 1:** Take sterile saline/thrombin-Tranfer sterile solution into the sterile liquid transfer cup. Draw up all the sterile solution from the sterile liquid transfer cup into empty syringe.



**Step 2:** Connect the syringes – Remove cap from SURGISPON® powder prefilled syringe and attach this syringe to the sterile syringe carrying sterile solution, through luer connector.



**Step 3:** Mix the contents of syringes – Begin mixing both by pushing the sterile solution into the SURGISPON® powder prefilled syringe. Continue exchanging the solution between syringes until all components are thoroughly mixed and the consistency is even.



**Step 4:** Attach the applicator tip for product to be ready for clinical use.

## Ordering Information

Product ID	Product Name	Weight/Volume	No. of Pieces Per Box
SSP-PS06KIT	POWDER KIT	550mg/6ml*	1
SSP-PS10KIT	POWDER KIT	1gm/10ml*	1

\* Deliverable volume after preparation

### Aegis Lifesciences Private Limited

215/216, Mahagujarat Industrial Estate,  
Ahmedabad, Gujarat-382213, India.

+91 9512503131 +91 9512603131

marketing1@aegis-lifesciences.com  
aegis.lifesciences@gmail.com

AL/BRC/SSPPS/01/R4 | Date: 08/2024



www.aegis-lifesciences.com