
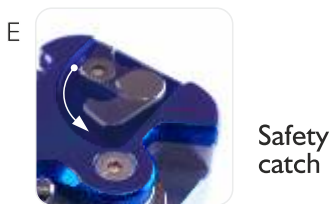


Biocut-N

Reusable automatic device
for soft tissues biopsy

Cod: Biocut-N

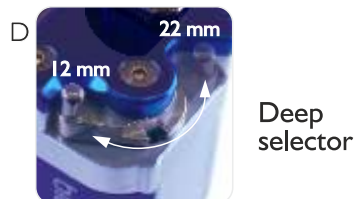

Biocut-N

Reusable automatic device
for soft tissues biopsy

Cod: Biocut-N

Instruction to use:



Open the cover
to insert PG
needle in the
device.


Close the
cover



When both
indicators are
white
Biocut-N
is charged



Push the
shooting
bottom

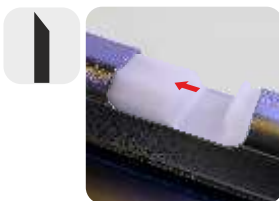


Use the charging
lever first time
to charge
the cannula



Select
shooting
deep

12mm
22 mm

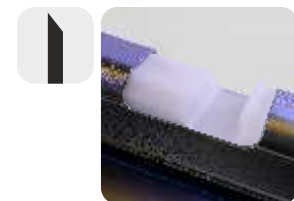

Use the
lever
to release
the sample



Use the
charging lever
second time
to charge
the stylet



Remove the
safety catch
before
shooting

To order:

CODE
BIOCUT-N

PACK
I



biomedical

Biomedical srl Via G.B.Lulli, 43 50144 Firenze Italy
Tel +39 055 352678 +39 055 332822 Fax +39 055 331629
www.biomedical-bio.net sales@biomedical-bio.net

Bone biopsy needles

Guillotine needles for biopsy on soft tissues

Cyto-aspiration needles

Different specialistic needles and sets

Loco-regional anaesthesia needles and sets

Thermic sheets